



# Ramp Safety Checklist

This checklist outlines ways to safely use and extend the life of your Amramp steel ramp

## For Manual Chairs

- Confirm both user and caregiver can use the ramp as installed.
- Caregivers should always push user up the ramp or back the user down the ramp. Use lap belts where available.

## For Power Chairs & Scooters

- Always use the lowest speed setting possible.
- Never reverse up or down the ramp. To change directions, proceed to the end of the ramp, turn around on a level surface, and then proceed forwards up or down the ramp.

## Rain, Ice, or Snow

- Rain, ice, frost, and snow may cause slippery conditions due to the slope of the ramp. Use added caution before using a ramp in these conditions.
- Do not use salt de-icer, snowblowers, or metal shovels on the ramp**
- To remove snow and ice, we recommend using a broom or plastic shovel only. Use only approved liquid de-icer (available locally from Amramp).

## Maintain Ramp Finish

- Normal use of the ramp and stair systems may cause scratches of the painted finish. Annual or twice annual touch up is recommended using an exterior, Rust-Oleum grade enamel spray. Alternatively, contact Amramp to arrange for service from your local owner.

## General Use Safety

- Ramp is designed for wheelchair or pedestrian use only. Do not use ramp for loading materials of any kind.
- Do not exceed 850 lb. of combined weight of user, wheelchair, and caregiver. Call Amramp if there are weight changes as proper use of the ramp may require modifications.
- Do not allow children to play on the ramp.
- Do not place electrical cords on the ramp.
- Keep the ramp clean and free of leaves, branches, debris, or other obstructions.
- Keep the ramp area lit to ensure full visibility after dark.
- Mesh surface may not be suitable for pets.
- Always contact Amramp before making any modifications to the ramp system.
- Do not use the ramp if it has been hit by a car, snow plow, or damaged in any way, which can cause misalignment. Call Amramp for repair.
- Per installation standards, ramp is anchored at top transition for optimal safety.
  - If anchoring is refused, please initial here: \_\_\_\_\_ accepting risk of the ramp potentially shifting away from the property.
- Periodically check the connection of the ramp to the property. Contact Amramp if adjustment is required.

**I have reviewed the above instructions and acknowledge that I have received the ramp system in good condition:**

**Client Name:** \_\_\_\_\_

**Signature:** \_\_\_\_\_

**Date:** \_\_\_\_\_

**Installer:** \_\_\_\_\_

For office use only: Maintain completed & signed copy with closed order # \_\_\_\_\_

**For Customer Service, please call 800-649-5215 or visit [www.amramp.com](http://www.amramp.com)**